


Firmware Update Instructions

Step 1: Flexineb E3 Controller → Move the **Red Switch** to RIGHT position for update function




Step 2: Connect a Mini-B USB / Standard USB Cable

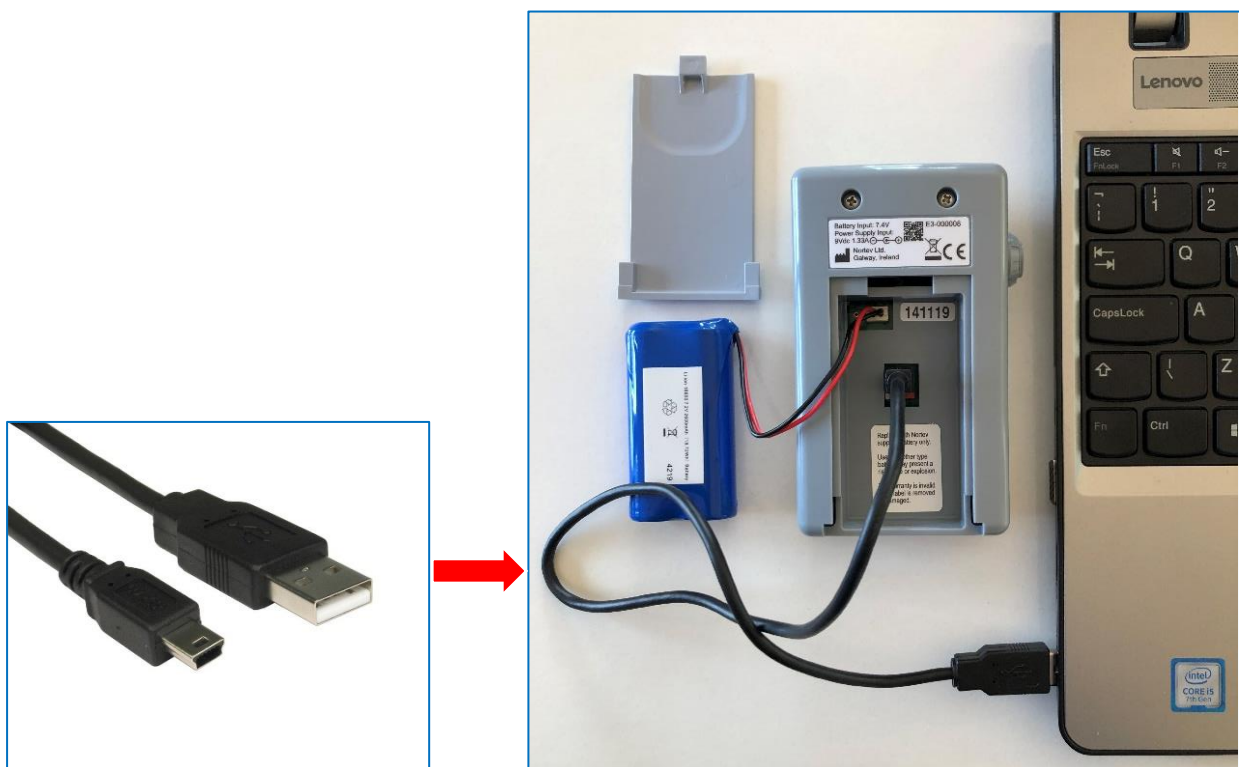


Firmware Update Instructions

Step 3:

Go to www.flexineb.com, **LOG IN** or **CREATE ACCOUNT** & follow the **Click on** signposts in Steps 3 to 10

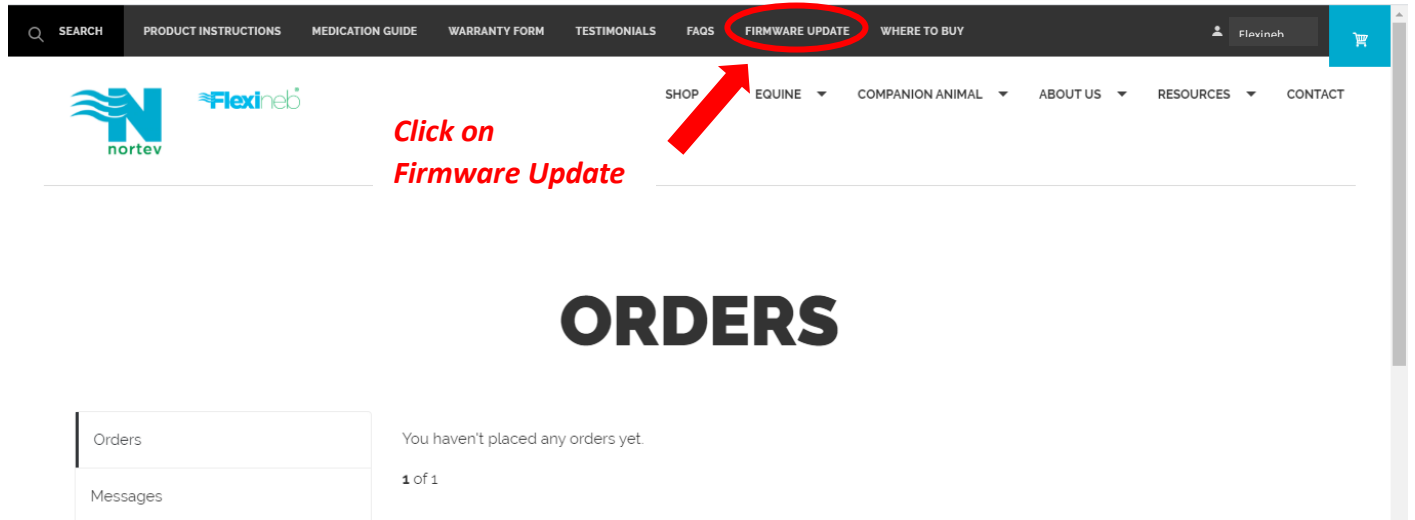

Step 4:

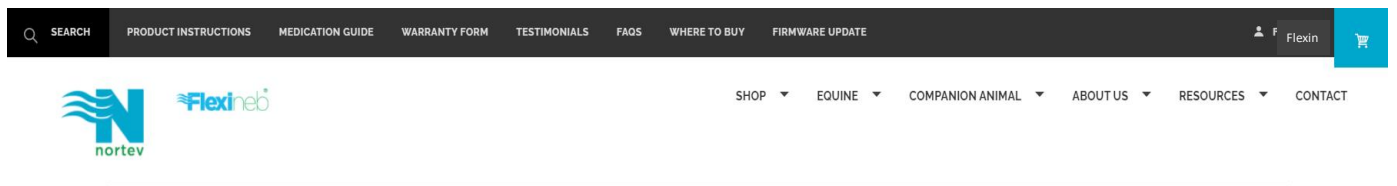

FLEXINEB® E3 FIRMWARE UPDATE

Date Modified: 24.08.2020

**Click on
Firmware
Update Version**

To upgrade your Flexineb® E3 unit, please read and follow the below instructions:

- [Firmware Download Instructions \(.pdf\)](#)

Then download and run the Firmware Update Utility Program:

- [Firmware Update Client - Version 0931 \(.exe\)](#)

Step 5:

SEARCH PRODUCT INSTRUCTIONS MEDICATION GUIDE WARRANTY FORM TESTIMONIALS FAQs WHERE TO BUY FIRMWARE UPDATE

SHOP ▾ EQUINE ▾ COMPANION ANIMAL ▾

FLEXINEB® E3 FIRMWARE UPDATE

Date Modified: 24.08.2020

To upgrade your Flexineb® E3 unit, please read and follow the below instructions:

- [Firmware Download Instructions \(.pdf\)](#)

Then download and run the Firmware Update Utility Program:

- [Firmware Update Client - Version 0931 \(.exe\)](#)

Click on downloaded FlexinebE3 Firmware.exe file

CONTACT US LEGAL SOCIAL SUBSCRIBE

Flexineb-E3-Firmw...exe

Step 6:

Windows protected your PC

Microsoft Defender SmartScreen prevented an unrecognized app from starting. Running this app might put your PC at risk.

[More info](#)

Click on More Info

Don't run

Step 7:

Windows protected your PC

Microsoft Defender SmartScreen prevented an unrecognized app from starting. Running this app might put your PC at risk.

Application: Flexineb-E3-Firmware-Update (1).exe
Publisher: Unknown publisher


Click on Run anyway

Run anyway Don't run

Firmware Update Instructions


Step 8:

Flexineb Programmer Utility. (Version 0.9.3.1 31 July 2020)

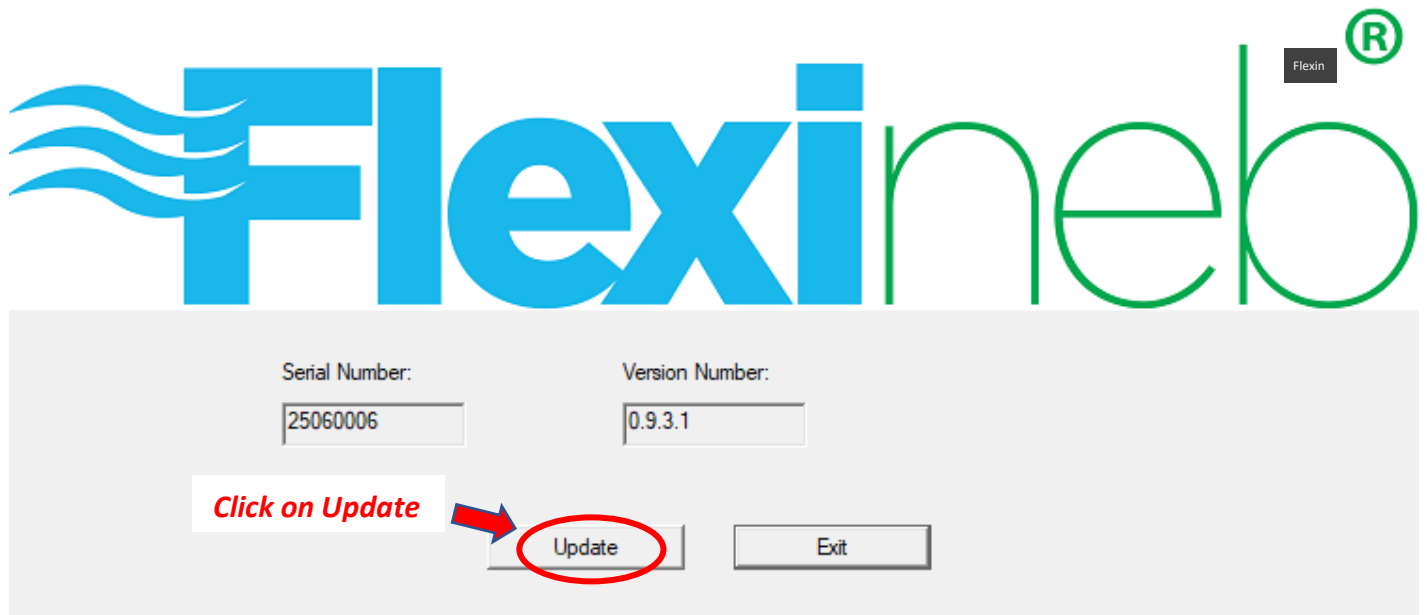

Step 9:


Flexineb Programmer Utility. (Version 0.9.3.1 31 July 2020)

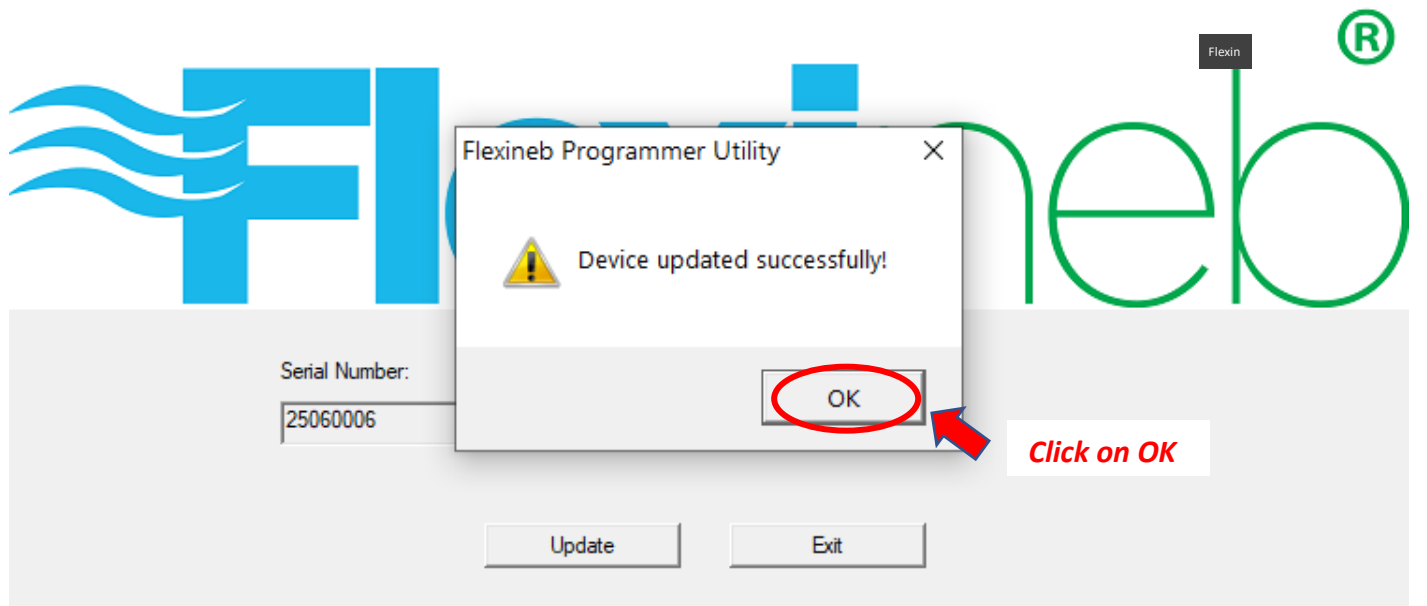

Firmware Update Instructions


Step 10:

Flexineb Programmer Utility. (Version 0.9.3.1 31 July 2020)

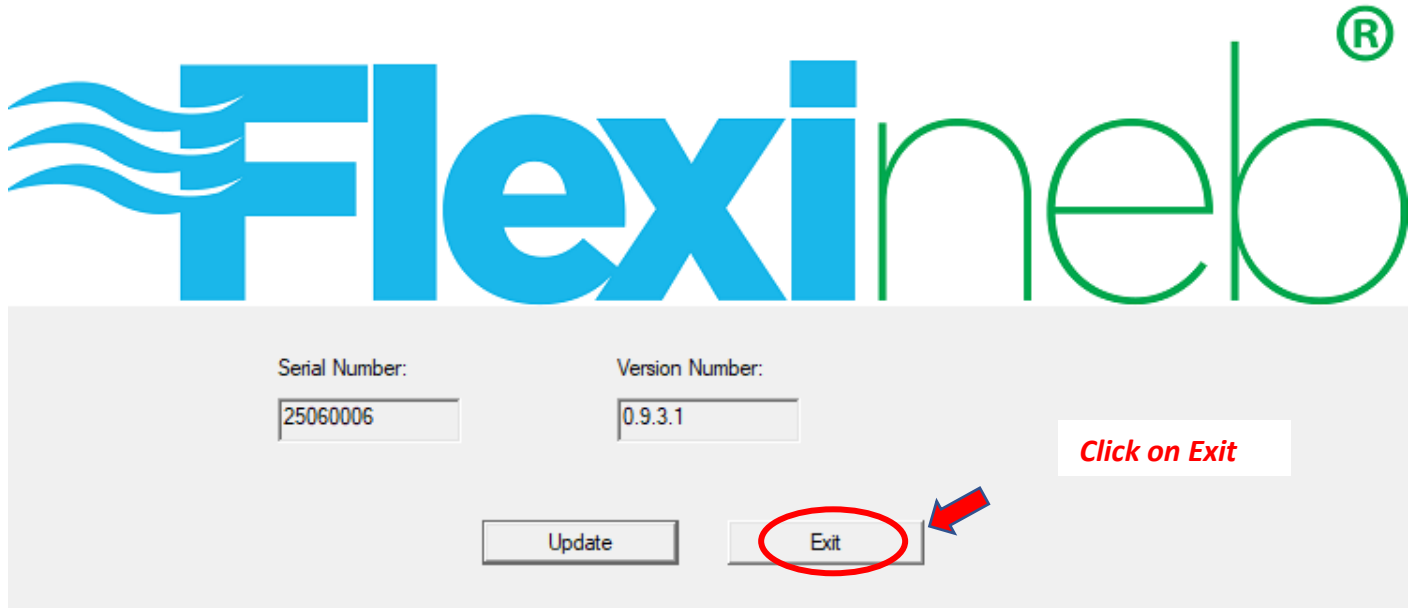

Flexin

Step 11: Flexineb E3 Controller → Move Red Switch back to LEFT position for normal function

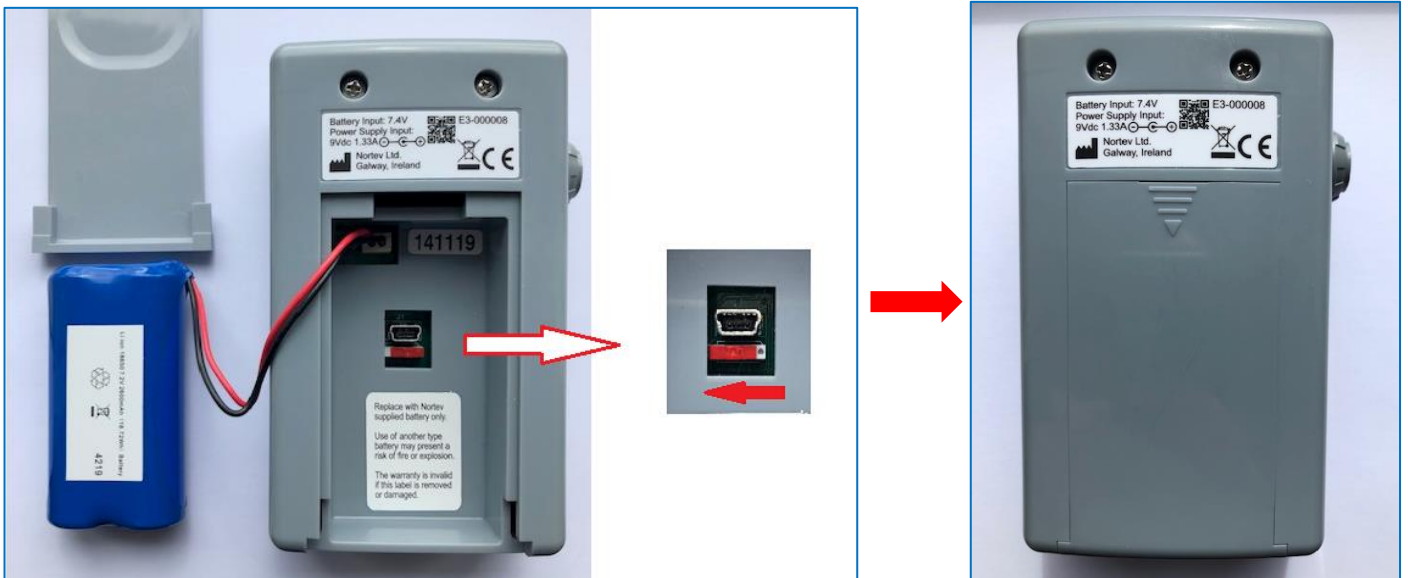

END