

Firmware Update Instructions


Step 1:

Remove the battery door and battery as shown. If your unit has a **Red Switch**, then move it to the RIGHT position for firmware update function.




Step 2:

Depending on the E3 version you have connect either a Mini-B USB cable (digital camera type cable) or Micro USB cable (android phone type cable) to the E3 controller end and connect the standard USB end to your laptop or PC as shown.

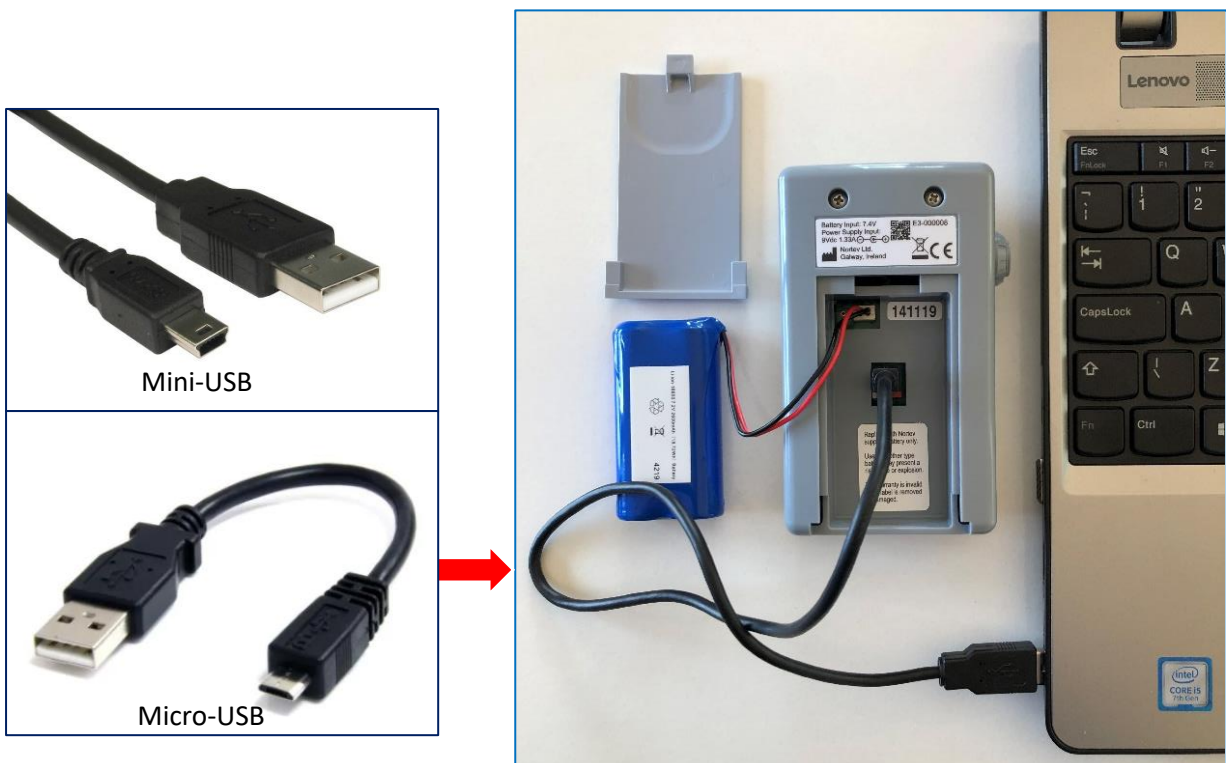

Step 3:

Go to www.flexineb.com, and click on the **FIRMWARE UPDATE** tab

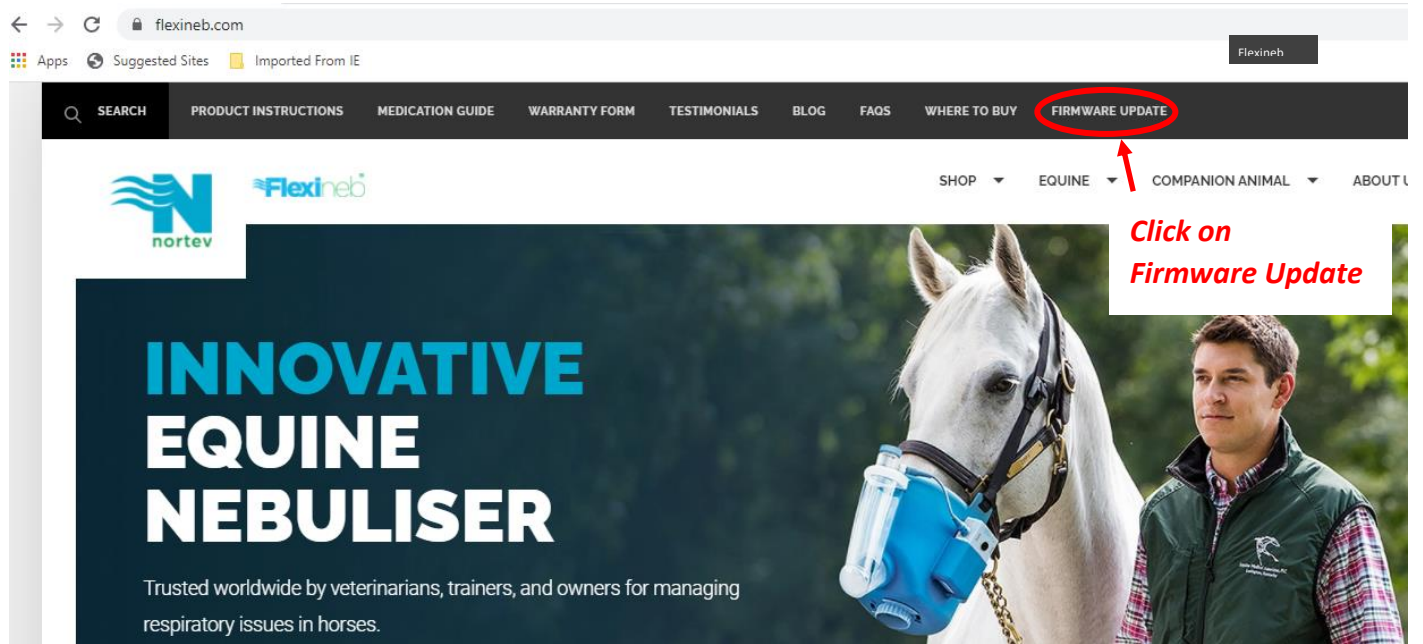

Step 4:

FLEXINEB® E3 FIRMWARE UPDATE

Date Modified: 24.08.2020


To upgrade your Flexineb® E3 unit, please read and follow the below instructions:

- [Firmware Download Instructions \(.pdf\)](#)


Then download and run the Firmware Update Utility Program:




Step 5:

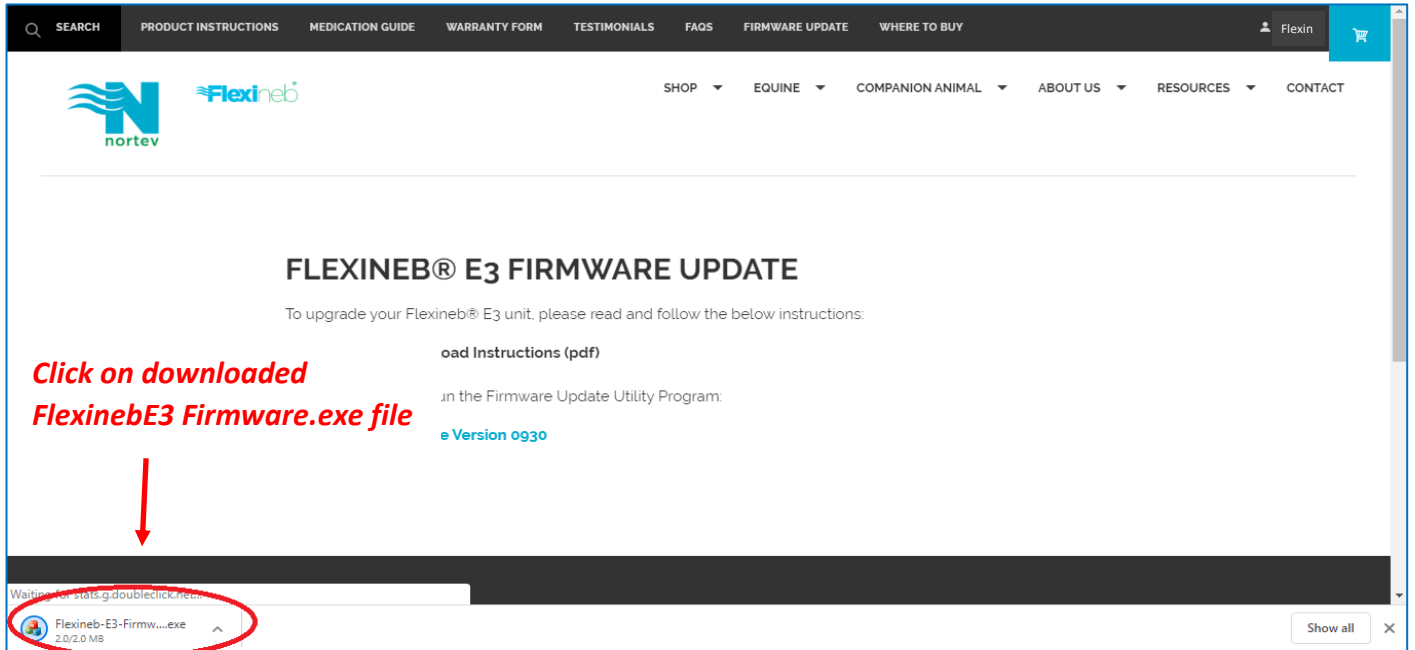

Step 6:

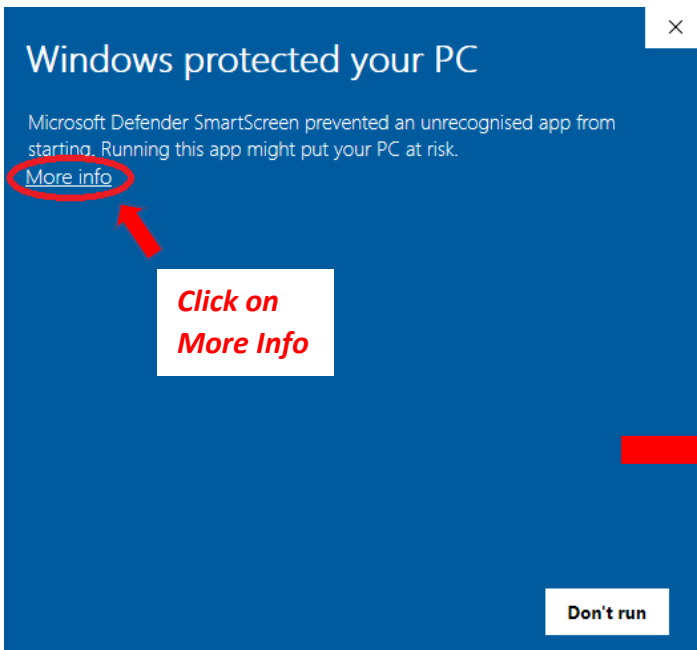


Step 7:

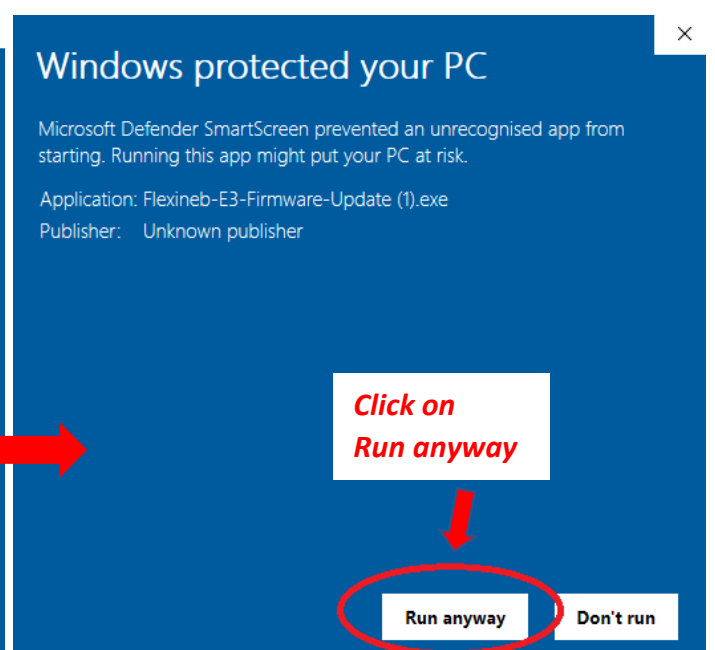


Firmware Update Instructions

Step 8:

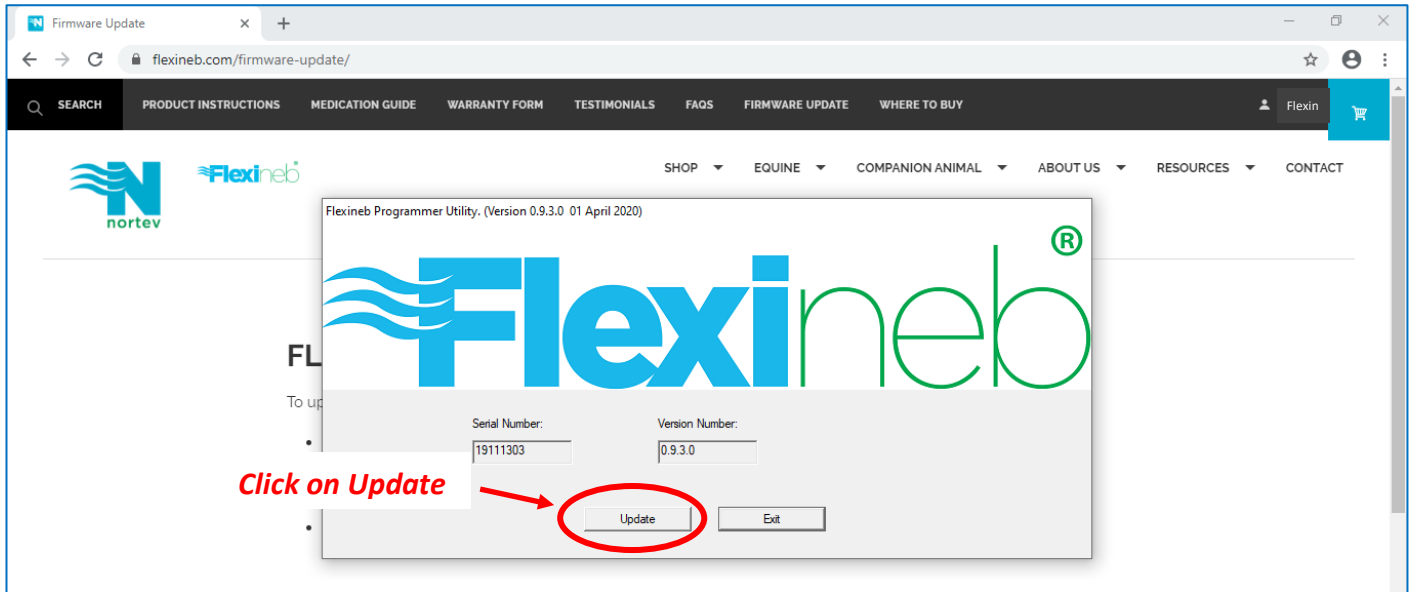

Step 9:

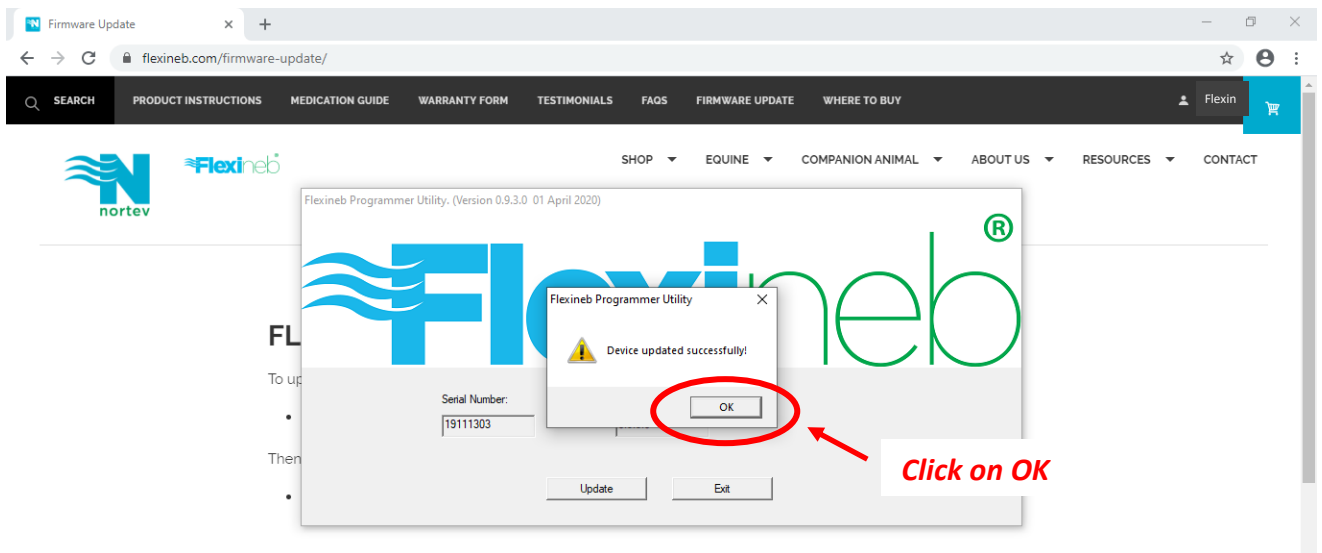

Step 10:




Firmware Update Instructions

Step 11:

Move **Red Switch** back to the LEFT position for normal function on units with this switch.

Re-fit the battery and add the battery door. All done.

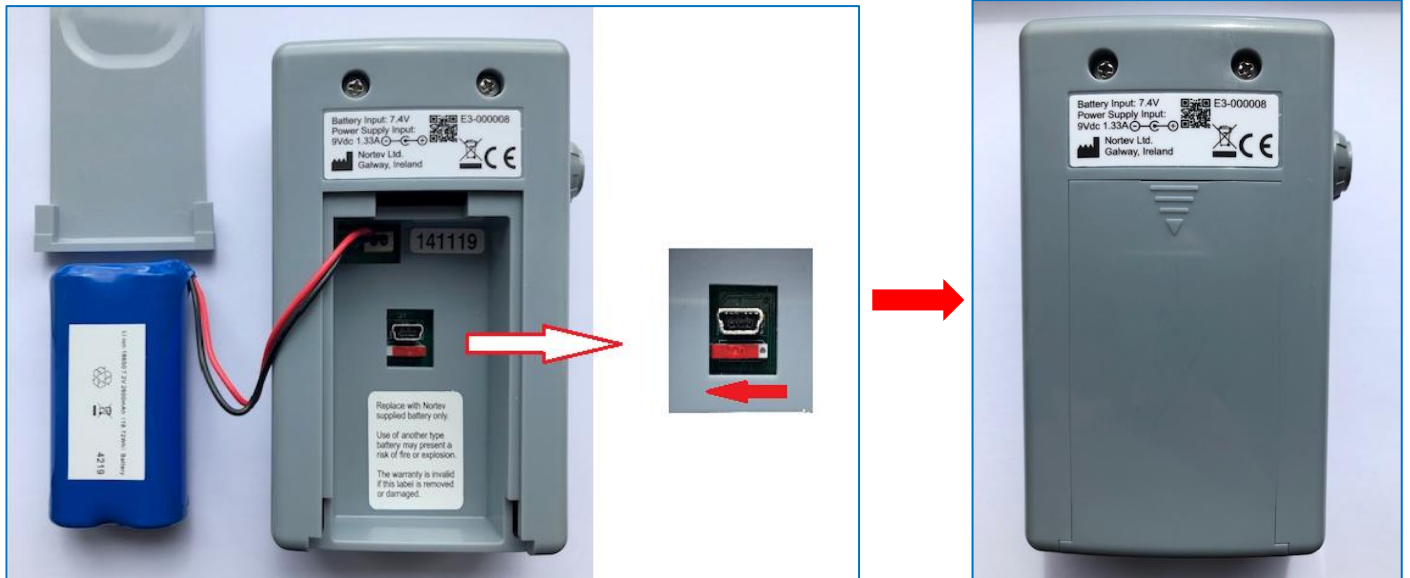

END